

Inspire nCPAP™

Non-invasive respiratory support



The complete respiratory care system



Inspire nCPAP™

Non-invasive respiratory support

Nasal CPAP for treating infants has developed a great deal in the past 20 years with many devices now being available.

The Inspire nCPAP™ range from Inspiration Healthcare builds on the experience of the past to allow care-givers a cost effective solution for supporting the respiratory compromised infant.

nCPAP works well when the Added Work Of Breathing is reduced ensuring the infants do not tire so quickly and allowing them to use energy to grow. Using established fluidic principles, the Inspire nCPAP™ range is more effective than traditional methods of delivering nCPAP.



nCPAP is proven to be beneficial in supporting infants with respiratory problems



Comprehensive range of prongs and masks



Simple fixation with bonnets to fit most infants

How the system works

Fresh gas is accelerated through small entrance tubes, it enters the internal chamber at a certain angle, allowing it to become unstable and to follow the path of least resistance. As the baby inhales, fresh gas is directed towards the infant.

As inspiration ends, the flow is flipped around and leaves the generator through the expiratory tube effectively creating active expiration.

The expiratory tube also acts as a reservoir of fresh gas, therefore should the infant's peak flow demand rise above the delivered flow, gas will be entrained from the expiratory tube, allowing the infant's peak flow requirement to be satisfied.



A cost-effective solution

Compatibility

The Inspire nCPAP™ range has been tested on market leading nCPAP drivers such as Infant Flow™ which deliver flows and pressures as shown in the graph below.

Any device that can deliver a flow of fresh gas in the range shown at a driving pressure greater than 120 cmH₂O will cause the fluidic logic in the generator to be effective. Inspiration Healthcare have a full list of compatible drivers available on request – if you have any doubts just email info@inspiration-healthcare.com

Because of the nature of the device, it will also operate in the same manner with 'bi-level' nCPAP drivers i.e FABIAN Therapy Evolution, as these simply alternate between flows at the same driving pressure.

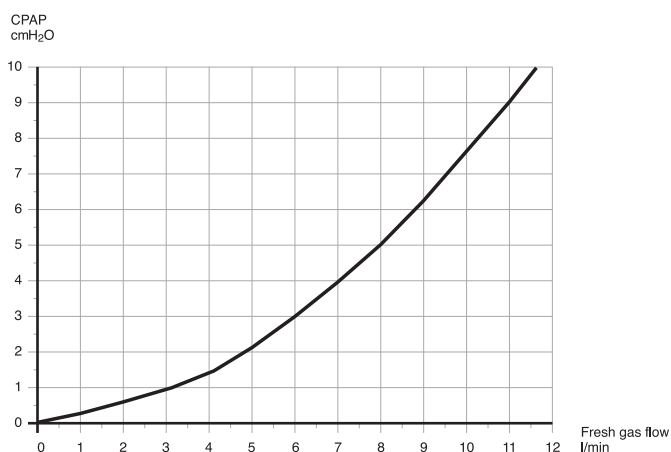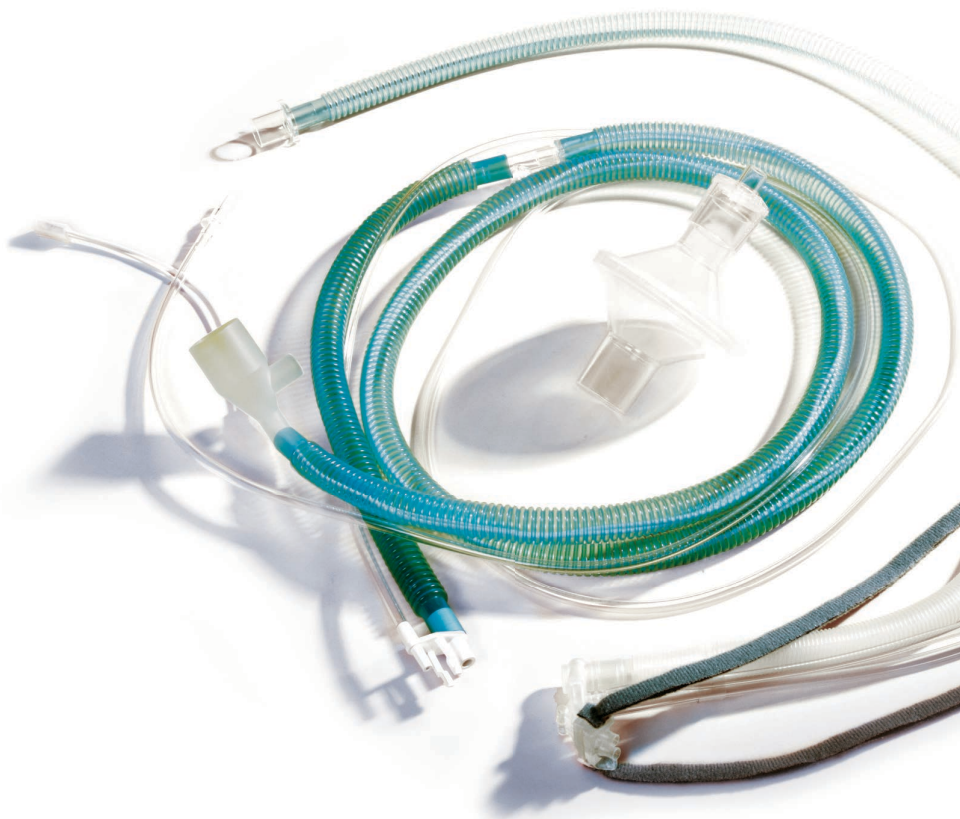

Clinical staff need to be assured that they will deliver the prescribed pressure from their devices. The graph above shows the relationship between the flow and pressure delivered by the Inspire generator.

Accessories

Inspire nCPAP™ from Inspiration Healthcare has a comprehensive range of accessories. Colour-coded bonnets, nasal prongs and nasal masks are available to ensure the generator is comfortable and securely placed on the infant. Nasal masks are available for patients who have been nasally intubated and therefore have asymmetrical nostrils or where an adequate seal cannot be maintained using the prongs. Circuits can be supplied in various configurations with the option of an autofeed humidification chamber.



Ordering Information

Generator and Circuits

Inspire nCPAP™ Generator complete with nasal prongs (Case of 10)	IHC600/10
Inspire nCPAP™ Generator only (Case of 10)	IHC616/10
Inspire nCPAP™ Silencer (Case of 20)	IHC602/20

Masks

Inspire nCPAP™ Mask Small (Case of 10)	IHC609/10
Inspire nCPAP™ Mask Medium (Case of 10)	IHC610/10
Inspire nCPAP™ Mask Large (Case of 10)	IHC611/10
Inspire nCPAP™ Mask Extra Large (Case of 10)	IHC612/10

Prongs

Inspire nCPAP™ Nasal Prong Extra Small (Case of 10)	IHC604/10
Inspire nCPAP™ Nasal Prong Small (Case of 10)	IHC605/10
Inspire nCPAP™ Nasal Prong Medium (Case of 10)	IHC606/10
Inspire nCPAP™ Nasal Prong Large (Case of 10)	IHC607/10

Bonnets

Inspire nCPAP™ White Bonnet 16-18 cm (Case of 10)	IHBWH000/10
Inspire nCPAP™ Grey Bonnet 18-20 cm (Case of 10)	IHBGY00/10
Inspire nCPAP™ Pink Bonnet 20-22 cm (Case of 10)	IHBPK0/10
Inspire nCPAP™ Brown Bonnet 22-24 cm (Case of 10)	IHBBR1/10
Inspire nCPAP™ Yellow Bonnet 24-26 cm (Case of 10)	IHBYE2/10
Inspire nCPAP™ Blue Bonnet 26-28 cm (Case of 10)	IHBBL3/10
Inspire nCPAP™ Gold Bonnet 28-30 cm (Case of 10)	IHBGO4/10
Inspire nCPAP™ Green Bonnet 30-32 cm (Case of 10)	IHBGR5/10
Inspire nCPAP™ Burgundy Bonnet 32-34 cm (Case of 10)	IHBBU6/10
Inspire nCPAP™ Orange Bonnet 34-36 cm (Case of 10)	IHBOR7/10
Inspire nCPAP™ Dark Green Bonnet 36-38 cm (Case of 10)	IHB DG8/10
Inspire nCPAP™ Navy Bonnet 38-40 cm (Case of 10)	IHBNA9/10

Circuits

For a full range of Inspire circuits please see our First Breath Circuit brochure.

The details given in this brochure are correct at time of going to press. The photo used on the front of this brochure is a retouched representation of the product. The images used are for illustration purposes only and may differ from the actual product. The company reserves the right to improve the products shown.

Not all accessories are available in all markets. Please contact your local Inspiration Healthcare representative for more information

500-132(1) 2020-01



Inspiration Healthcare Limited

2 Satellite Business Village, Fleming Way,
Crawley, West Sussex, RH10 9NE
United Kingdom

T +44 (0)1455 840 555

F +44 (0)1455 841 464

E info@inspiration-healthcare.com

W inspiration-healthcare.com



THE QUEEN'S AWARDS
FOR ENTERPRISE:
INTERNATIONAL TRADE
2019