



# Resmon™ Pro V3

the new Forced Oscillation Technique



## THE RESMON PRO V3, THE NEW FOT SYSTEM, NOW MADE EVEN EASIER

Following the success of the Resmon™ PRO FULL, we introduce the Resmon™ Pro V3 - a higher level of FOT. This new advanced device uses the same superior measurement calculation algorithms and reporting with redesigned user interface and faster electronics. It has never been easier to accurately and non-invasively evaluate patients with COPD, adult and pediatric Asthma as well as airways clearance technique in physiotherapy. All it takes is a few normal tidal breaths.

The Resmon Pro V3 uses the forced oscillation technique (FOT) to measure the mechanical properties of the lung and airways. FOT easily assesses the respiratory system's response to acoustic waveforms sent to the opening of the airway into the lungs at different frequencies. The Resmon Pro V3 is an essential tool for all patients including individuals who cannot perform spirometry, elderly, pediatric and those with neuromuscular diseases.



The Resmon Pro V3 can provide a simple, effort-independent way to detect airways obstruction and localizing it as peripheral, central or heterogeneous. During the test, the Resmon Pro V3 will measure small and central airways function and also assess tidal expiratory flow limitation (EFL) in adults while aiding in bronchial reversibility testing or trending it over time for therapeutic monitoring.

With such versatility, the Resmon Pro V3 is ideal for virtually any testing environment—including pulmonary laboratories, at the bedside, private offices, on-site school or workplace settings and sites for clinical trials or research studies.



## RESMON PRO V3 SYSTEM

*DESIGNED, DEVELOPED, PRODUCED AND SUPPORTED BY EXPERTS*

The Resmon Pro V3 is the result of more than 15 years of research conducted in cooperation with key scientific, clinical and research centers in Europe, US and Australia.

### GREAT FEATURES FOR CLINICAL AND RESEARCH USE

- “WITHIN-BREATH” ANALYSIS OF BREATHING PATTERNS AT FAST SAMPLING FREQUENCY (200 HZ)
  - Real-time display of resistance (Rrs), reactance (Xrs) and flow
  - Accurate measurement of inspiratory, expiratory and total parameters
  - Full respiratory pattern measurement and reporting of VE, VT, RR, Ti/Ttot, Vt/Ti, Vt/Te
- THREE MEASUREMENT MODES:
  - Innovative “enhanced optimized” multi-frequency mode of 5-11-19 Hz
  - Single-frequency mode options of 5, 6, 8, 10 Hz (for children, severely obstructed patients and special research purposes)
  - “Enhanced optimized” Pseudo Random Noise (PSN) of 5-37 Hz
- EXCLUSIVE “10 ACCEPTED BREATHS” MODE WITH AUTOMATIC END-OF-TEST
  - Comfortable testing for the patient with an automatic discard of non-physiological and non-coherent breaths, choice of number of accepted breaths, 10, 15, 20 etc. to end the test automatically, minimizing operator intervention.
- AUTOMATIC STIMULUS ADJUSTMENT
  - Automatic adjustment of stimulus amplitude based on patient impedance to maximize measurement quality (and for optimal patient comfort)
  - CV % (coefficient of variation) within-measurement and within-session (with multiple trials capability), Z-score for predicted normal value % calculation
- MINIMAL DEAD SPACE OF ONLY 35CC
- PATENTED, AUTOMATIC DETECTION AND QUANTIFICATION OF EXPIRATORY FLOW LIMITATION (EFL) WITH  $\Delta XRS$  INDEX GRAPH AND % OF FLOW LIMITED BREATHS (FL%), ALSO USED FOR PRE-POST BRONCHODILATOR OR PRE-POST TREATMENT EVALUATION (PATENT NR. WO2003103493).
- NEW Slow Vital Capacity testing (NOT available in the US, not FDA approved)
  - SVC for monitoring of restrictive patterns
  - IC for hyperinflation detection, pre-post testing for evaluating effect treatment



## CONVENIENT SIZE AND FUNCTIONS

- Self-contained and compact unit with password access, built-in PC, with data transfer on USB memory stick and directly to PCs via USB-OTG
- Easily transportable allowing for testing at the bedside, in the lab and at off-site locations (school, office, etc.)
- Quiet operation with automatic “active-wash” silent fan for expired CO<sub>2</sub> removal from the system
- Suitable for children, 35 ml dead space
- Unique impedance verification tube included, to daily check the accuracy of both resistance (Rrs) and reactance (Xrs) before the first daily trial using an automatic, software driven procedure.
- Wide 10.1 in (25.7 cm) high-resolution color touchscreen for fast, easy and intuitive test management
- Built-in database to store and retrieve patient data
- Multi-user access with password protected data security
- Detailed report of results for further trending, analysis and statistics Data available in RAW waveform data, XSL/CSV and PDF output
- Connectivity options, USB and HDMI ports

## PUBLICATIONS

The Resmon Pro has been used in a wide selection of publications for clinical and research uses in asthma, COPD and evaluation of airways clearance therapies. Updated list available upon request.

## SPECIFICATIONS

FLOW MEASUREMENT	Mesh type	PHYSICAL DIMENSIONS	Height: 12.2 in (31 cm)
RANGE	±1.5 L/s		Width: 11.4 in (29 cm)
LINEARITY	±2%		Depth: 10.2 in (26 cm)
MOUTH PRESSURE	Range: ±25 cm H <sub>2</sub> O	WEIGHT	14.7 lbs (6.4 kg) with support arm
MOUTH PRESSURE LINEARITY	0.05% full scale		Device only 9.4 lbs (4.3 Kg)
RESOLUTION	0.015 cm H <sub>2</sub> O		

TESTING "WITHIN BREATH"  
SIGNALS MODE

SINGLE FREQUENCY  
5, 6, 8, 10 Hz

MULTIPLE FREQUENCY  
5 + 11 + 19 Hz

PSEUDO RANDOM NOISE (PSRN)  
5-37 Hz

MEASUREMENT ACCURACY

FOR IMPEDANCE PARAMETERS  
≤ ± 0.1 cmH<sub>2</sub>O/L/s or ≤ ± 9%  
of the measured value

FOR BREATHING PATTERN PARAMETERS  
≤ 10% of the measured value

FOR VOLUME PARAMETERS  
≤ ±100 mL or ≤ ±3.5% of the  
measured value

PRESSURE ± 100 PA  
ENVIRONMENTAL SENSOR\* Temperature: ±1°C

Humidity: ±3% (relative humidity)

*\*NOT AVAILABLE IN US NOT FDA APPROVED*

CALIBRATION Factory calibration according to international recommendations + auto-zeroing of the sensors before each measurement

Calibration check with a test object (supplied with the device) and with a 3L calibration syringe (not supplied with the device), required for the measurement of slow spirometry volumes (not available in the US, not FDA approved).

PATIENT LOAD 0.55-0.69 cm H<sub>2</sub>O/L/s in the frequencies of normal breathing (0.1-1 Hz)  
DEAD SPACE 35 mL

CONNECTIVITY 2 full-speed (2.0) USB ports | 1 USB-OTG  
Ethernet 10/100/1000  
HDMI

PROCESSOR AND MEMORY Dual core architecture, including a Cortex™-A9 1GHz processor, 1GB RAM, 8 GB flash memory  
DISPLAY 10.1 in (25.7 cm) HD color display with multi-touch capacitive touchscreen and anti-glare film (can be used with medical gloves)

ELECTRICAL SPECIFICATIONS Medical grade 100/240 V, 50/60 Hz 60 W input AC/15 VDC output power supply (included)

STAND-BY CURRENT 500 mA

AVERAGE CURRENT 1500 mA

MATERIALS EXTERNAL CASE ABS

SUPPORT ARM Aluminum

LOUDSPEAKER MEMBRANE Silicone rubber

CERTIFICATION European MDD 93/42 - CE | US FDA (K152585) | Australia TGA | Health Canada

DESIGNED, DEVELOPED AND MANUFACTURED BY:

**RESTECH**  
RESPIRATORY TECHNOLOGY

[www.restech.it](http://www.restech.it)

MGC DIAGNOSTICS CORPORATION, manufactured by RESTECH, distributed exclusively by Medical Graphics Corporation  
350 Oak Grove Parkway St. Paul, Minnesota USA 55127-8599

© 2020 MGC Diagnostics Corporation or one of its affiliates. All rights reserved.

All specifications subject to change without notice. Products may vary from those illustrated.

MGC Diagnostics Corporation and its affiliates are equal opportunity/affirmative action employers committed to cultural diversity in the workforce.

Part# 060168-001 RevA

CE 0426